ACCU-FINE®

An ACCU-CHEK® product

EN Sterile, single-use pen needles

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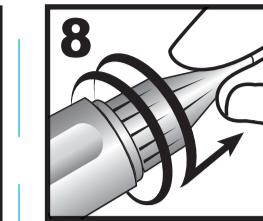
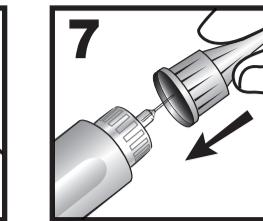
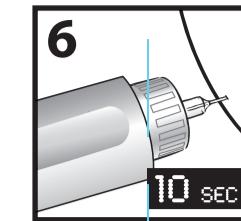
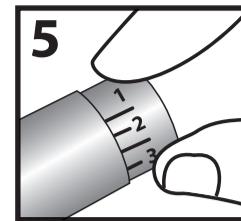
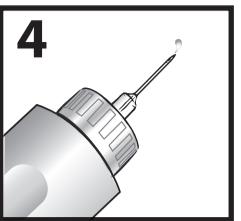
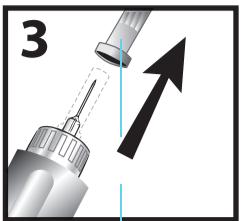
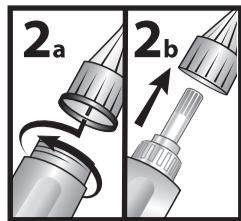
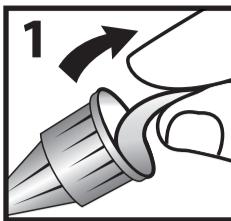
EN Instructions for use

1. Remove the seal from the pen needle.
2. Push the pen needle straight on the pen injector device and twist until it is tight (2a). Do not push the pen needle at an angle. Then remove the outer protective cap and set it aside (2b). Do not discard. You will need it later to remove the pen needle from the pen injector device.
3. Remove the inner protective cap by pulling it straight out to prevent bending of the needle and/or damaging the needle tip.
4. Prime your pen injector device as per your pen's instructions for use to prevent incorrect dosage caused by air in the cartridge or a clogged needle. If no drug flow is seen after several attempts, attach a new pen needle.
5. Set your drug dose.
6. Make the injection as directed by your healthcare professional, do not bend the needle. Inject slowly. Do not withdraw needle immediately after injection. Do not change the direction of the pen needle while it remains in the body as this can result in bending or breaking of the needle.
7. Replace the outer protective cap.
8. Remove the pen needle from the pen injector device after each injection to prevent air from entering the cartridge and drug leaking out.

Warning: Remove both the outer protective cap and the inner protective cap before an injection. If both the outer cap and the inner protective cap are not removed before use, the medication or dose may not be injected, which may result in serious injury or death.

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EN References obtained from ISO 15223 Medical devices — Symbols to be used with medical device labels, labeling and information to be supplied.

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